

Steril-Eze® Horizontal Open Surgical Case Carts

ASSEMBLY INSTRUCTIONS



CAUTION

INSPECT CONTENTS IMMEDIATELY AND FILE CLAIM WITH DELIVERING CARRIER FOR ANY DAMAGE. **SAVE YOUR BOX AND ALL PACKING MATERIALS.** YOU ARE RESPONSIBLE FOR DAMAGE TO YOUR UNIT IF RETURNED IMPROPERLY PACKED.


Thank you for your purchase. Before assembly, read this instruction sheet carefully. After reading, be sure to store this instruction in a safe place, should it be needed for future reference.



Parts List

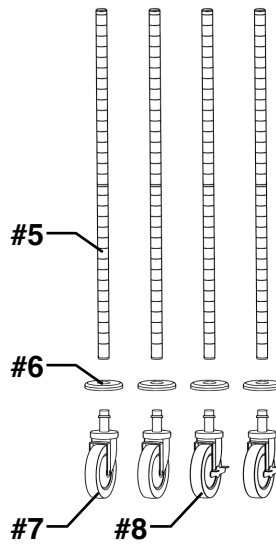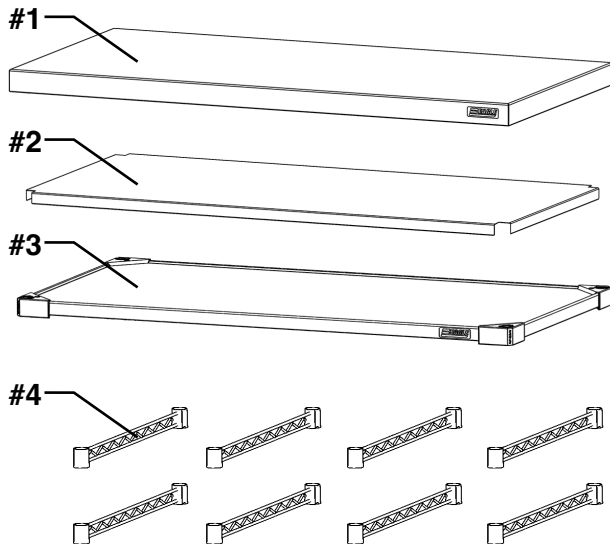

#	qty	description
#1	1	top
#2	1	center shelf
#3	1	solid shelf
#4	8	hanger rail
#5	4	post
#6	4	donut bumper
#7	2	caster
#8	2	caster with brake
#9	6	set of four pairs of split sleeves

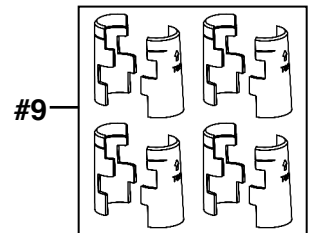

(continued on back page)


EG10122 Revised 05/24

Steril-Eze® Horizontal Open Surgical Case Carts

ASSEMBLY INSTRUCTIONS


Step 1 - Assemble casters and bumpers to posts.

- Slide bumper onto bottom of post.
- Insert caster stem into bottom of post, and tap with mallet or hammer. Several blows usually required.
- Repeat on remaining casters/bumpers/posts.

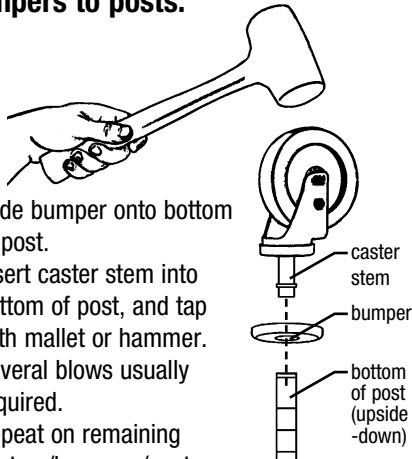


Step 2 - Attach bottom shelf to posts.

- Snap the plastic split sleeves onto each post at the second groove from the bottom.
- Note: The wider part of the split sleeve is the bottom.*
- Lower the bottom shelf onto split sleeves.

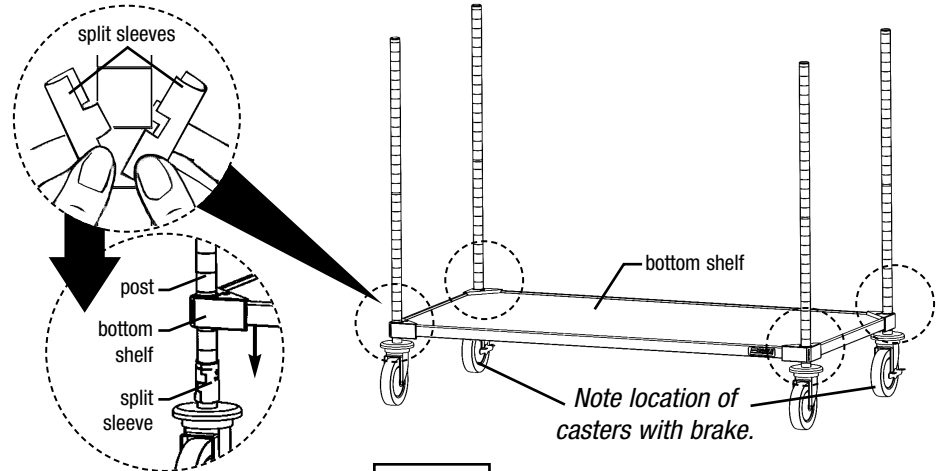



Step 3 - Attach hanger rails, working from bottom to top.

- Starting with bottommost hanger rails, snap the plastic split sleeves onto each post (see illustration at right for suggested heights).
- Note: The wider part of the split sleeve is the bottom.*
- Lower the hanger rail onto split sleeves.
- Repeat this step to attach all hanger rails.

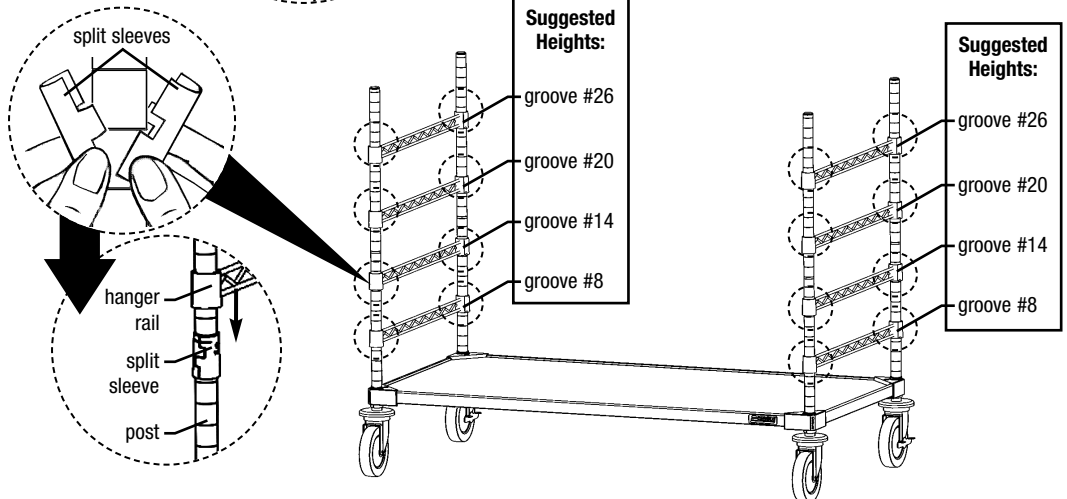

Step 4 - Attach center shelf.

- Lay shelf across 2nd tier of hanger rails, as shown.
- Note: Shelf is removable and can easily be moved to the tier of your choice.*

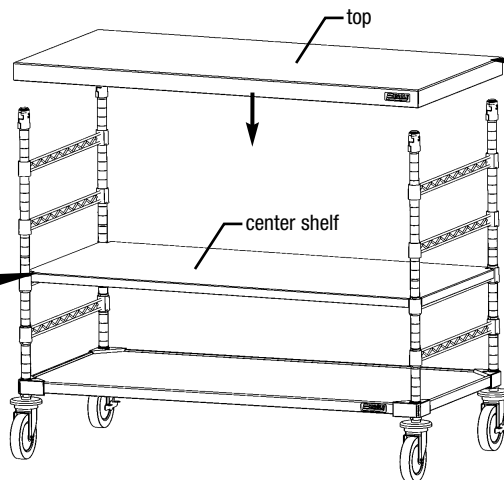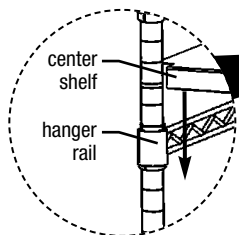

Step 5 - Attach top.

- Snap the plastic split sleeves onto the top groove of each post.
- Note: The wider part of the split sleeve is the bottom.*
- Lower the top onto split sleeves.



Assembly is complete.